

ORDERING INFORMATION

Ordering Number	Product or Service	Description	Units (Per Box)
D2201-003	Rezūm Delivery Device	<ul style="list-style-type: none">• Sterile Delivery Device with cable and tubing• Sterile Syringe• Sterile Spike Adaptor• 50 ml Sterile Water Vial	box of 5
D2202-003	Rezūm Accessory Kit for Delivery Device	<ul style="list-style-type: none">• Sterile Syringe• Sterile Spike Adaptor• 50 ml Sterile Water Vial	box of 5
G2200-003	Rezūm Generator	<ul style="list-style-type: none">• Portable Generator• Power Cord• 1-Year Manufacturer's Warranty	1
G2205-002	Rezūm System Extended Warranty – Gold Plan	<ul style="list-style-type: none">• 1-Year Extended Warranty	
G2206-002	Rezūm System Extended Warranty – Platinum Plan	<ul style="list-style-type: none">• 2-Year Extended Warranty	

Other materials that are typically required for the Rezūm System procedure include, but are not limited to, the following items:

- Saline supply at room temperature (1L recommended)
- 4mm, 30 degree, 30cm Storz or Richard-Wolf rigid cystoscope lens
- Light source and cord
- 240V socket

Please see the Rezūm System Instructions for Use for the full list of materials.


KEBOMED

KEBOMED UK | Unit 7 Kingsford Rural Business Centre | Kentisbeare | Devon | EX15 2AU

the new wave in BPH treatment

rezūm®
water vapor therapy


Using Convective WAVE™ Technology, Rezūm is a safe, effective¹ and modern method for the treatment of BPH.

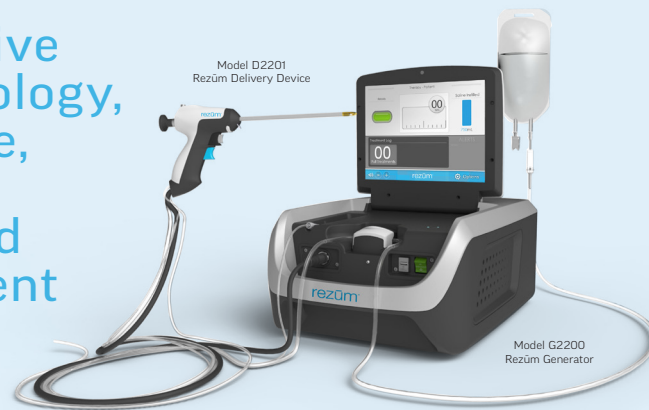

Each 9 second treatment uses 0.42ml of radiofrequency heated sterile water vapor which rapidly and uniformly disperses through the tissue interstices.

The condensation of water vapor releases stored thermal energy. Cell membranes are gently denatured, thereby causing immediate cell death, the vasculature is closed, and there is denervation of the alpha adrenergic nerves within the treatment zone providing you with an efficient, uniform, predictable treatment.

Post-operatively, this ablated tissue is resorbed by the body's natural healing response, reducing the volume of tissue allowing the prostatic urethra to open relieving LUTS.

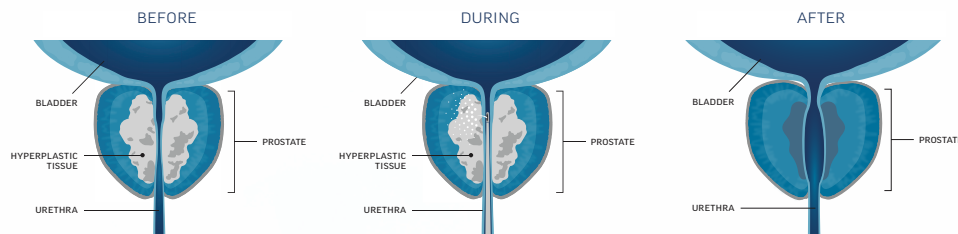

Key benefits of Rezūm therapy:

- Fast, effective and, precise BPH treatment¹
- Ablates targeted tissue, which is then resorbed
- Treats enlarged central zones and/or median lobes
- Clinically proven, safe and effective¹
- Preserves sexual and urinary functions¹
- Rapid return to normal activities¹

The appropriate Rezūm patients are the following:

- Men who have chosen not to take BPH medications, are taking them but are dissatisfied, or have discontinued them
- Men who are not receptive to surgical BPH procedures
- Men who are unable to have general anesthesia

Indications

The Rezūm System is indicated to relieve symptoms, obstructions, and reduce prostate tissue associated with BPH in men with:

- Prostates $\geq 30 \text{ cm}^3$
- Prostates with hyperplasia of the central zone and/or a median lobe


Contraindications

The use of the Rezūm System is contraindicated for patients:


- With a urinary sphincter implant
- Who have a penile prosthesis

Key Efficacy Results

IPSS outcomes are closely replicated across treatment arm, crossover and pilot study¹ subjects.

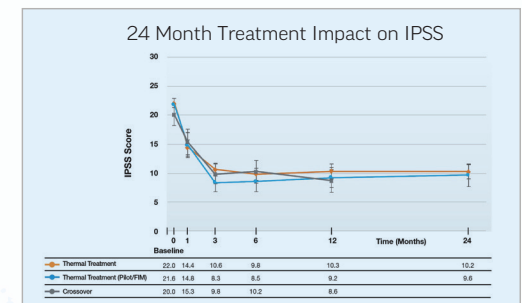

THERMAL THERAPY ARM

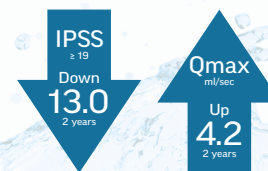

Rezūm demonstrated durable outcomes at 2 years with an IPSS improvement of 11.2 (-51%) and flow rate improvement of +4.2 ml/sec.¹

- Subjects with IPSS ≥ 19 had mean IPSS improvement of -13.0 at 2 years.¹

¹ Roehrborn CG, Gange SN, Gittelman MC et al. Convective radiofrequency water vapor thermal therapy with Rezūm System: Durable two-year results of randomized controlled and prospective crossover studies for treatment of lower urinary tract symptoms due to benign prostatic hyperplasia. *J Urol* 2017. In Press.