



Innovation in Gynaecology



LiNA Xcise™

Cordless Morcellator

Improved Safety


Reducing procedure time



Adjustable coreguard



Reducer cap

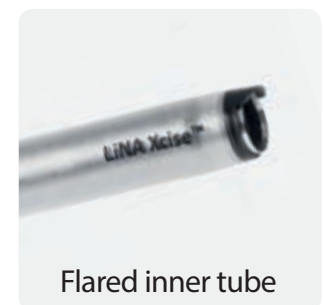

Flared inner tube

The LiNA Xcise™ is a cordless fully disposable morcellator intended for tissue morcellation during laparoscopic gynaecological procedures. The unique design of the LiNA Xcise™ provides safe and easy control during hysterectomy and myomectomy.

LiNA Xcise™ Cordless Morcellator

Simplicity

Single use device that is completely ready to use

- **Cordless and no assembly required**
The morcellator can be used right out of the box and as it is cordless there is no need for assembly to driver box or cable. The LiNA Xcise™ is therefore quick, simple and easy to use and facilitates unlimited mobility of the OR staff.
- **Fully disposable**
The LiNA Xcise™ is RoSH compliant and fully disposable in regular hospital waste.
- **Power**
The LiNA Xcise™ has a higher torque and runs at 1,000RPM to morcellate through the most fibrotic tissue (Largest reported uterus morcellated by LiNA Xcise was 2930gr).



Innovation

Easy and effective morcellation

- **Flared cutting tube**
Cutting tube on The LiNA Xcise™'s flares out to a larger diameter as tissue is pulled through, which reduces drag, provide long tissue strips and reduces tissue rotation.
- **Inner bevel blade design**
The LiNA Xcise™ features a large inner bevel blade design preventing the blade from going dull from inadvertent contact with the tenaculum.
- **Adjustable coreguard**
The coreguard on the cutting tube can be adjusted and rotated 360° according to the hand position ensuring optimal peeling effect.
- **5mm reducer cap**
The reducer cap has a simple snap-on design for quick exchange and optimal use of 5mm instrument.



Safety

Full control during morcellation

- **Three-step positioning**
The LiNA Xcise™ has three blade positions to secure an optimal and safe morcellation;
"Safety Guard" – Blade fully retracted into the cannula for patient safety
"Cut 1" for surface peeling
"Cut 2" for coring



Ordering information:

LiNA Xcise™

REF.	Product	Packing
MOR-1515	LiNA Xcise – 15mm morcellator with obturator – single use	2x boxes of 6, sterile
MOR-1515-1	LiNA Xcise – 15mm morcellator with obturator – single use	2x boxes of 6, sterile, individually packed
MOR-1515-6	LiNA Xcise – 15mm morcellator with obturator – single use	1x box of 6, sterile