

# LiNA AntiFog Solution™

Provides clear camera optics during  
endoscopic surgery



# LiNA AntiFog Solution™

LiNA Medical's range of antifog solutions consists of LiNA Elfred™, LiNA Clear Sight™ and LiNA Clear Sight Wipe™.

## Functional. LiNA Elfred™

LiNA Elfred™ consists of a small sponge in an aluminium foil container with adhesive on the back. The sponge is prefilled with 6ml antifog solution. LiNA Elfred™ offers a high degree of convenience as it is simple and ready for use:

- 1) Remove the adhesive strip under the container and attach it to the patient drape or instrument table.
- 2) Remove the lid of the container.
- 3) Press the scope onto the sponge and twist it to apply a membrane of the antifog solution.
- 4) If required, the sponge can be taken out of the container to clean the sides of the scope.

The LiNA Elfred™ is available with or without alcohol.

## LiNA Clear Sight™

LiNA Clear Sight™ consists of a 6ml bottle and a sponge with adhesive on the back. LiNA Clear Sight™ is easy to use:

- 1) Attach the sponge to the patient drape or instrument table.
- 2) Open the bottle and pour solution onto the sponge.
- 3) Press the scope onto the sponge and turn it to apply a membrane of the antifog solution.

The LiNA ClearSight™ is available with or without alcohol.

## LiNA Clear Sight Wipe™

LiNA Clear Sight Wipe™ is prefilled with antifog solution and individually packed in a peel pouch. The LiNA Clear Sight Wipe™ is only available without alcohol.



### Ordering Information:

LiNA AntiFog Solution™ **CE 0459**

Product code	Description	UOM
CS-20	<b>LiNA Elfred™</b> – 6 ml of AntiFog Solution for endoscopic optics. Sterile, single use.	Box of 30 ea
CS-20A	<b>LiNA Elfred™</b> – 6 ml of AntiFog Solution for endoscopic optics with 4% alcohol. Sterile, single use.	Box of 30 ea
CS-30	<b>LiNA Clear Sight™</b> – 6 ml of AntiFog Solution for endoscopic optics. Sterile, single use.	Box of 20 ea
CS-30A	<b>LiNA Clear Sight™</b> – 6 ml of AntiFog Solution for endoscopic optics with 4% alcohol. Sterile, single use.	Box of 20 ea
CS-40	<b>LiNA Clear Sight Wipe™</b> – Sterile tissue with AntiFog Solution for endoscopic optics. Single use.	Box of 30 ea

©2021 LiNA Medical ApS. All rights reserved.  
LiNA is a registered trademark of LiNA Medical in EU, US and CN.

LiNA Medical ApS

• Formervangen 5  
DK-2600 Glostrup  
Denmark

• Tel.: +45 43 29 66 66  
Fax: +45 43 29 66 99  
info@lina-medical.com

• www.lina-medical.com  
CVR: 10054974

Quality Management System  
EN ISO13485:2016

