


# MiniONE<sup>®</sup> Button

## *Low Profile Balloon Gastrostomy Tube*

### Replacing AMT MiniONE<sup>®</sup> Balloon Button\*

To ensure a proper fit, it is crucial to measure the stoma length using the AMT Balloon Stoma Measuring Device (BSMD). This is not a substitute for the DFU/IFU (Directions for Use/Instructions for Use). 

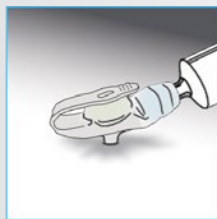

#### DEFLATE BALLOON

Deflate the existing device using a Luer syringe or Luer Lock syringe.



#### REMOVE DEVICE

Remove the device, gently pull up on the device with slow and steady pressure.



#### PREPARE KIT

Prepare the new device, apply a water soluble lubricant to the device.



#### INSERT DEVICE

Insert the device into the stoma site, until it is flush against the skin. Remove the blue stiffener.



#### FILL BALLOON

Fill the balloon to the recommended fill volume, which is printed on the balloon fill valve.



# MiniONE® Button

## *Low Profile Balloon Gastrostomy Tube*

**ENFIT®**

Available MiniONE® Family Sizes: Balloon 12F – 24F



MiniONE® also available in Non-Balloon Options:

MiniONE® Capsule Non-Balloon: 14F, 18F, 20F, 24F. MiniONE® Non-Balloon: 14F, 18F, 20F, 24F.

## Feed Sets Included in Kit

